
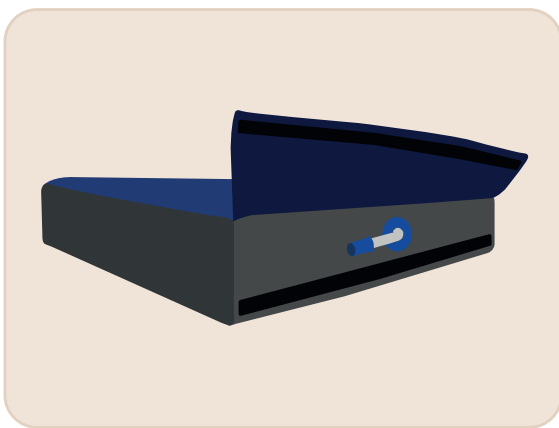



Adjustment of seating comfort:

Soft protective cover with hook-and-loop fastener

Handle for easy carrying

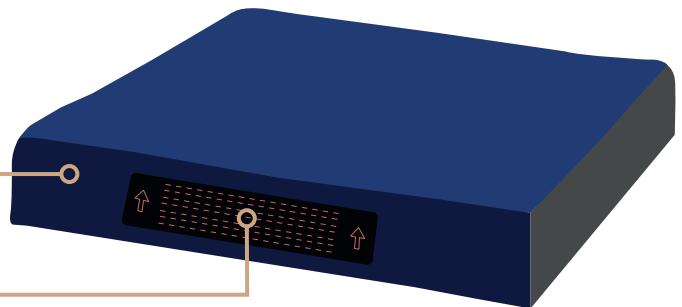


1. Lift the protective cover.

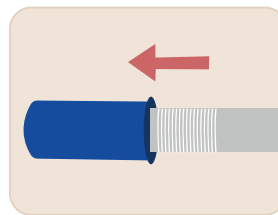


2. Open the valve by pulling the lock and wait approx. 30 seconds, until the air-cells have inflated automatically.


The valve lock cannot fall off.

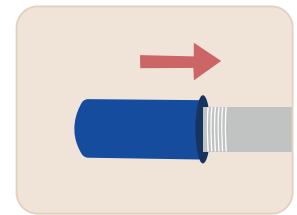


3. Close the valve by pressing.

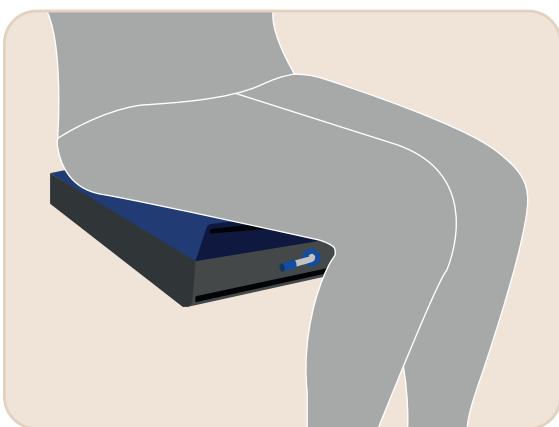

4. Put the cushion on the wheelchair or seat. The handle must point to the front and the attached arrows must point upwards. Sit on the cushion.

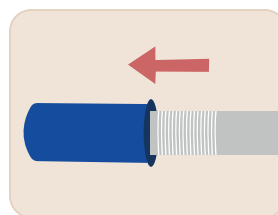

5. Carefully open the valve again until the ideal sitting comfort is achieved.

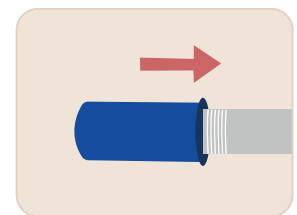

6. Close the valve and the protective cover.